

# FKG XP-endo<sup>®</sup> + Race<sup>®</sup> EVO



QR > Race<sup>®</sup> EVO



QR > XP-endo<sup>®</sup>

Optional  
Race<sup>®</sup> EVO  
25/.08



K File  
ISO 10



Race<sup>®</sup>EVO  
15/.04



XP-endo<sup>®</sup> Shaper  
3D Preparation




Optional  
XP-endo<sup>®</sup> Finisher  
Cleaning



Optional  
Race<sup>®</sup> EVO  
40/.04



Optional  
Race<sup>®</sup> EVO  
50/.04

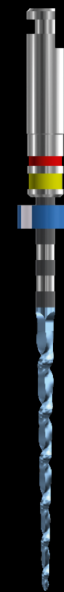

**DIFFICULT CASES**

800 - 1000/min

60 sec, 1000/min

**LARGE CANALS**

REFER TO FKG USER MANUAL [WWW.FKG.CH](http://WWW.FKG.CH)

Link for videos: [www.ftcdental.ch/fkg-race-evo](http://www.ftcdental.ch/fkg-race-evo)

# Obturation .02, .04, .06



**Sterile Paper  
points**



**Total Fill®  
BC points**



**Total Fill® BC  
sealer**

REFER TO FKG USER MANUAL [WWW.FKG.CH](http://WWW.FKG.CH)



Link for videos: [www.ftcdental.ch/fkg-race-evo](http://www.ftcdental.ch/fkg-race-evo)